




LAPAROSTAT™

Strength and Agility with the Touch of a Hand

CIVCO's Laparostat provides an extra hand for the laparoscopic surgeon.

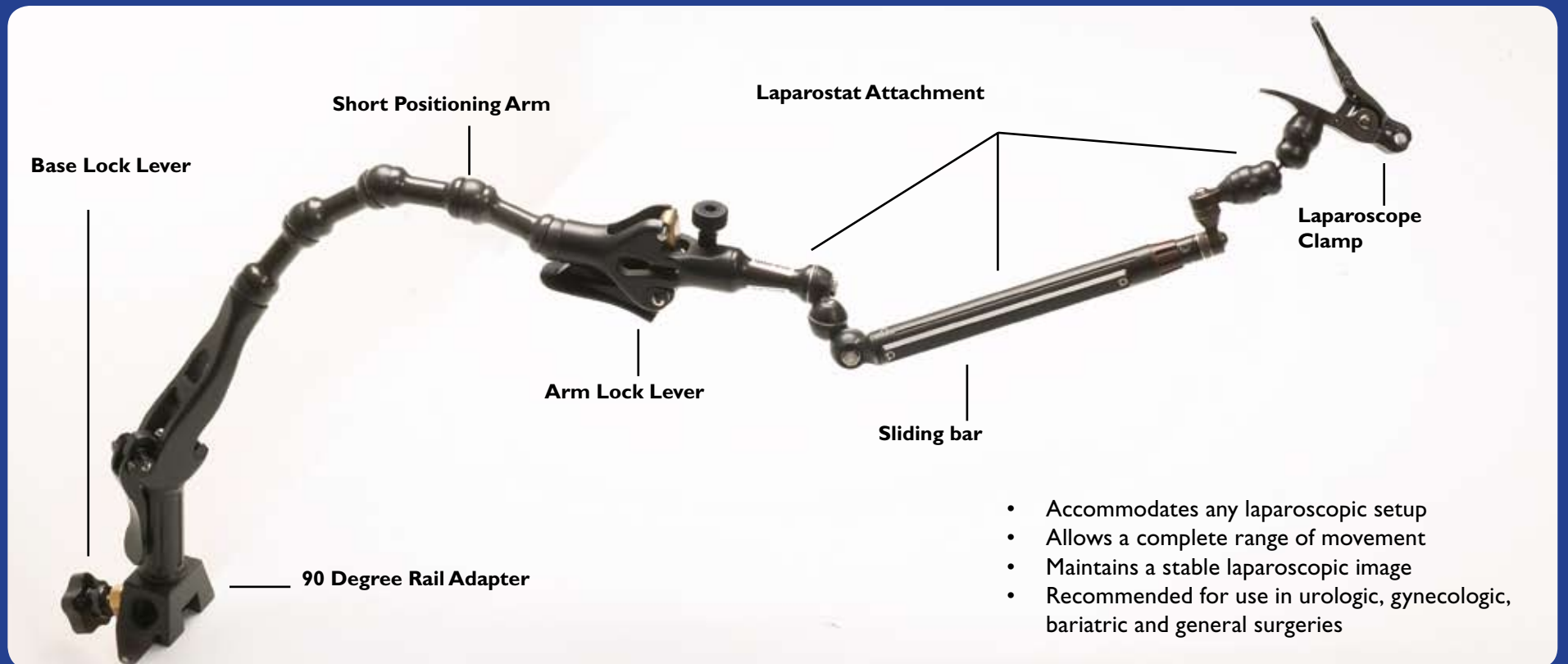


Simple three step setup:

1



- Attach 90 Degree Rail Adapter to surgical table rail
- Any of three mounting ports can be used to ensure optimal positioning
- Device can be attached anywhere along surgical table

2

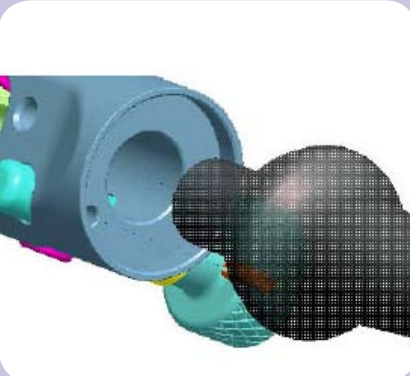

- Attach to Positioning Arm using large and small pins and tightening knob
- Thread pins together

3



- Position Laparostat vertically and drape arm with the sterile CIV-Flex™ cover
- Once draped, proceed with final positioning
- **Sterility:** CIV-Flex sterile covers are designed to cover the entire system and allow access to knobs and levers. These pliable and durable covers will keep the system clean and enable full function of the Laparostat in a sterile field.
- **Cleaning:** The arm and components are not sterile and may be cleaned by wiping with alcohol and common hospital germicidal solutions. The device may squeak during normal use and should not be lubricated. Do not Autoclave, soak or immerse any components as this will cause damage not covered by warranty.

Ask your Clinical Coordinator for the Laparostat!



800.445.6741 | 319.656.4447
WWW.CIVCO.COM

Worldwide Headquarters: 102 First Street South, Kalona, Iowa 52247-9589 USA
COPYRIGHT © 2007, CIVCO IS A REGISTERED TRADEMARK OF CIVCO MEDICAL SOLUTIONS. LAPAROSTAT AND CIV-FLEX ARE TRADEMARKS OF CIVCO. NOT ALL PRODUCTS MAY BE LICENSED IN ACCORDANCE WITH CANADIAN LAW. PRINTED IN USA. PATENT PENDING. 2007P-2220.